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This document contains the basic 'Instructions for Use' of Materialise 3-matic Medical Software. Detailed instructions can be found in the 'User Guide'. English is the language used in the 3-matic Medical software interface.

1 Instructions for Software Use

Description and performance characteristics of 3-matic Medical software

Materialise 3-matic Medical is a software that combines CAD tools with pre-processing (meshing) capabilities. To do this, it works on triangulated (STL) files which makes it suitable for freeform 3D data, like the anatomical data coming from the segmentation of medical images (from Mimics Medical). As such, 3-matic Medical can be seen as a tool that allows Anatomical CAD or what Materialise has coined 'Engineering on Anatomy'. 3-matic Medical may be used as a medical device, within the limits of the described below intended use statement.

3-matic Medical may be used to import anatomical data and other 3D objects in STL format or in numerous CAD formats including: CATIA, IGES, STEP, etc. Once objects have been loaded, they can be used for many applications of 'Engineering on Anatomy' including measuring, designing, modeling and 3D printing.

The modules that can be obtained for 3-matic Medical enable the users to do thorough 3D measurements and analyses, design an implant or surgical guide, or prepare the mesh for finite element modeling.

US – Indications for use

3-matic Medical is intended for use as software for computer assisted design and manufacturing of medical exo- and endo-prostheses, patient specific medical and dental/orthodontic accessories and dental restorations.

EU & ROW – Intended purpose

3-matic Medical is intended for use as software for computer assisted design and manufacturing of medical exo- and endo-prostheses, patient specific medical and dental/orthodontic accessories and dental restorations.

Warnings and recommendations

3-matic Medical software should only be used by trained Medical professionals. To engage in the experience, we recommend that all users participate in a training offered by a Materialise expert.

As stipulated in the End User License Agreement, Materialise warrants during the Warranty Period that 3-matic Medical shall operate substantially in accordance with the functional specifications in the Documentation. Any other warranty, whether express or implied, including but not limited to any warranty regarding fitness for purpose, safety, performance or effectiveness aspects of the medical application or device for which 3-matic Medical is used, is disclaimed.

3-matic Medical displays measurements with 4 digits behind the decimal point. Accuracy of measurements is limited by the data source.

Python scripts written by the user should always be validated by the user before being used for any purpose. Python scripts written for previous versions of 3-matic Medical should be re-validated by the user before being applied with the present version.

For a patient or user in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices): if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Best-practice tips and precautions

Be advised that the quality and resolution of your triangle mesh can influence the result of design or analysis operations. For best results, we recommend that industry standards are referenced and followed.

Instructions for use

Installing 3-matic Medical

Start the installation by launching the downloaded installation file (*.msi format). If this triggers the User Account Control to ask permission to make changes to your device, click 'Yes'. After this, the Setup Wizard will guide you through the installation. In every next dialog box, select your preferred options and choose 'Next' to continue. Do the actual installation by choosing 'Install', a progress bar will keep you up to date about the status.

If the application is installed successfully, the Finish dialog will appear, allowing you to complete the installation (click 'Finish').

If a new major, minor or bugfix release is distributed, installation is done in the same way as described above. The installer file will automatically overwrite (in case of a bugfix release) or install next to the already available versions. When starting the new version, you can choose whether or not to import the settings from the previous version.

Starting 3-matic Medical

After successfully installing the software on your PC, double click the 3-matic Medical icon on your desktop. You may also click on the **Start Button** → **All Programs** → **Materialise** → **3-matic Medical** to begin. Be sure to select the *Medical edition* for clinical applications.

Materialise software is protected by a key file. When you start 3-matic Medical for the first time or when your key has expired, the Key Request Wizard will automatically start up to assist you with the registration process. Available options are explained in the **Reference Guide**. We recommend that you select 'instant activation' in the Key Request Wizard. This will automatically renew your license Key File when it expires as long as you have internet access*.

*Other qualifications may apply.

Minimum system requirements

Software	Hardware
Windows® 10 – 64bit	Fourth generation Intel® Core™ i7 or equivalent

Version 1909	
Microsoft Edge® or equivalent	16 GB RAM
PDF viewer	DirectX® 12.0 compliant AMD Radeon/NVIDIA® GeForce® graphics card with 2 GB RAM
.NET framework 4.6.1 (or higher)	40 GB free hard disk space
	Resolution of 1920x1080 or higher

It is recommended to use our software within a hardware and/or network environment in which cyber security controls have been implemented including anti-virus and use of firewall. It is also recommended not to leave the workstation on which 3-matic Medical is running, unattended and to ensure the workstation is password-protected.

Description of Symbols

	Legal Manufacturer
R _x Only	Federal law (USA) restricts this device to sale by or on the order of trained medical professionals
	Caution, please consult the instructions for use and the accompanying documentation
	Consult instructions for use or consult electronic instructions for use
	Date of manufacture
	Batch code
	Medical device
	Unique Device Identifier
	Authorised representative in Switzerland

IFU for Materialise 3-matic Medical is supplied in electronic form in PDF format on www.materialise.com/electronic-instructions-for-use. Paper version may be requested by

emailing leaflet@materialise.be and will be provided within 7 calendar days at no additional cost.

Manufacturer contact detail

Manufactured by:

Materialise NV
Technologielaan 15
3001 Leuven, Belgium
Phone: +32 16 39 66 11
<http://www.materialise.com>

CH authorised representative:

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

1639

3-matic Medical is a CE-marked product.

Australian Sponsor:

Emergo Australia; Level 20, Tower II Darling Park; 201 Sussex Street; Sydney NSW 2000; Australia

This document is issued in JUN 2022.

© 2022 – Materialise N.V. All rights reserved.

Materialise, the Materialise logo and the Mimics and 3-matic product names are trademarks of Materialise NV.

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