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# Instructions for Software Use

## Description and performance characteristics of Mimics Medical software and plugins

Materialise Mimics Medical, briefly 'Mimics Medical', is a software tool for visualizing and segmenting medical images (such as CT and MRI) and rendering 3D objects. Mimics Medical may be used as a medical device, within the limits of the described below intended use statement.

Mimics Medical may be used to load and process stacks of 2D images from numerous formats including DICOM 3.0 format, BMP, TIFF, JPG and raw images. Once images are processed, they can be used for numerous applications of Engineering on Anatomy including measuring, designing, modeling and 3D printing.

Mimics Medical enables users to control and correct the segmentation of medical images such as CT and MRI-scans. A trained software user can easily and accurately define 3D models, which can then be used in Mimics Medical for visualization, measuring, treatment planning and producing an output file to be used for additive manufacturing (3D printing). Mimics Medical also has functionality for linking to third party software packages.

Mimics Medical can be extended with additional features via plugins. Mimics Medical plugins may be used as a medical device, within the limits of the intended use of Mimics medical.

Mimics Medical is not intended to predict the performance of a medical device.

## US – Indications for use

Mimics Medical is intended for use as a software interface and image segmentation system for the transfer of medical imaging information to an output file. Mimics Medical is also intended for measuring and treatment planning.

The Mimics Medical output can be used for the fabrication of physical replicas of the output file using traditional or additive manufacturing methods.

The physical replica can be used for diagnostic purposes in the field of orthopedic, maxillofacial and cardiovascular applications.

Mimics Medical should be used in conjunction with expert clinical judgement.

## EU & ROW – Intended purpose

Mimics Medical is intended for use as a software interface and image segmentation system for the transfer of medical imaging information to an output file. Mimics Medical is also intended for measuring and treatment planning.

The Mimics Medical output can be used for the fabrication of physical replicas of the output file using traditional or additive manufacturing methods.

The physical replica can be used for diagnostic purposes in the field of orthopedic, maxillofacial, and cardiovascular applications.

Mimics Medical should be used in conjunction with expert clinical judgement.

## Contraindications

Please note that Mimics Medical is not intended for mammography imaging.

## Warnings and recommendations

Mimics Medical should only be used by trained medical professionals. To engage in the experience, we recommend that all users participate in a training offered by a Materialise expert.

As stipulated in the End User License Agreement, Materialise warrants during the Warranty Period that Mimics Medical shall operate substantially in accordance with the functional specifications in the Documentation. Any other warranty, whether express or implied, including but not limited to any warranty regarding fitness for purpose, safety, performance or effectiveness aspects of the medical application or device for which Mimics Medical is used, is disclaimed. The output of (semi-)automatic segmentation functionality in Mimics Medical should always be verified by the user before being used for any purpose.

Inter-slice and in-slice distance of less than 1mm are recommended for treatment decisions based on 3D reconstructions for pre-operative planning or 3D printing.

Mimics Medical displays measurements with 4 digits behind the decimal point. Accuracy of measurements is limited by the pixel size of the images.

Python scripts written by the user should always be validated by the user before being used for any purpose. Python scripts written for previous versions of Mimics Medical should be re-validated by the user before being applied with the present version.

For a patient or user in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices): if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

## Best-practice tips and precautions

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

Mimics Medical can also be used by biomedical engineers or personnel equivalent by training or experience. Their results should be used in conjunction with expert clinical judgement.

Be advised that the quality of medical images determines the accuracy of the 3D model in Mimics Medical. Scanning protocols are left to the discretion of the user; however, we recommend that industry standards are referenced and followed. Only images obtained less than six months before should be used for planning and/or evaluating treatment options.

Please be aware that the DICOM coordinate system is reset for each DICOM series originating from all Hitachi scanners and from Siemens Symphony, Sonata, and Harmony scanners with software versions older than Syngo MR B13.

## Instructions for use

### Installing Mimics Medical

Start the installation by launching the downloaded installation file (\*.msi format). If this triggers the User Account Control to ask permission to make changes to your device, click 'Yes'. After this, the Setup Wizard will guide you through the installation. In every next dialog box, select your preferred options and choose 'Next' to continue. Do the actual installation by choosing 'Install', a progress bar will keep you up to date about the status.

If the application is installed successfully, the Finish dialog will appear, allowing you to complete the installation (click 'Finish').

If a new major, minor or bugfix release is distributed, installation is done in the same way as described above. The installer file will automatically overwrite (in case of a bugfix release) or install next to the already available versions. When starting the new version, you can choose whether or not to import the settings from the previous version.

### Starting Mimics Medical

After successfully installing the software on your PC, double click the Mimics Medical icon. You may also click on the **Start Button → All Programs → Materialise Software → Mimics Medical** to begin. Be sure to select the *Medical edition* for clinical applications.

Materialise software is protected by a key file. When you start Mimics Medical for the first time or when your key has expired, the Key Request Wizard will automatically start up to assist you with the registration process. Available options are explained in the **User Guide**. We recommend that you select 'instant activation' in the Key Request Wizard. This will automatically renew your license Key File when it expires as long as you have internet access.

### Installing Mimics Medical Plugins

First check if Mimics Medical is installed and activated properly. From the Materialise website download the plugin and run the installer. After completing the installation wizard successfully, launch Mimics. You can verify correct installation by going to **Help → About** and check if the plugin is listed in the about box. Before use, consult the user guide of the plugin available under **Plugin Guides** menu.

Some plugins may require separate activation, go to the key request wizard to initiate the activation process.

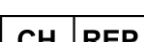
### Minimum system requirements

Software	Hardware
Windows® 10 – 64bit version 1909	Third generation Intel® Core™ i7 or equivalent
Microsoft Edge® or equivalent	16 GB RAM
PDF viewer	DirectX® 12.0 compliant AMD Radeon/NVIDIA® GeForce® card with 2 GB RAM
.NET framework 4.6.1 (or higher)	20 GB free hard disk space
	Resolution of 1920x1080 or higher

Note: Other qualifications may apply. When working with datasets larger than 1GB the system should comply with the minimum system requirements. Advanced segmentation tools such as Smart Expand, and Coronary segmentation require hardware as specified in the minimum requirements even for smaller datasets. When working with 4D or multi stack data, the amount of RAM needed increases as you import more image series into the project. To use Mimics Viewer service for auto segmentation or case sharing, minimum internet connection speed of 2Mbps is required. When using CT Heart automatic segmentation, CPU must be compatible with AVX2 (Advanced Vector Extensions).

It is recommended to use our software within a hardware and/or network environment in which cyber security controls have been implemented including anti-virus and use of firewall. It is also recommended not to leave the workstation on which Mimics Medical is running, unattended and to ensure the workstation is password protected.

## Description of Symbols

	Legal Manufacturer
	Caution, please consult the instructions for use and the accompanying documentation
	Consult instructions for use or consult electronic instructions for use
	Date of manufacture
	Batch code
	Medical device
	Unique Device Identifier
	Authorised representative in Switzerland

<b>R<sub>x</sub> Only</b>	Federal law (USA) restricts this device to sale by or on the order of trained medical professionals
---------------------------	---

IFU for Mimics Medical is supplied in electronic form in PDF format on [www.materialise.com/electronic-instructions-for-use](http://www.materialise.com/electronic-instructions-for-use). Paper version may be requested by emailing [leaflet@materialise.be](mailto:leaflet@materialise.be) and will be provided within 7 calendar days at no additional cost.

## Manufacturer contact details

### Manufactured by:

Materialise NV  
Technologielaan 15  
3001 Leuven, Belgium  
Phone: +32 16 39 66 11  
<http://www.materialise.com>

1639

Mimics Medical is a CE-marked product

Mimics plugins are class I medical device software

Australian Sponsor: Emergo Australia; Level 20, Tower II Darling Park; 201 Sussex Street;  
Sydney NSW 2000; Australia

### CH authorized representative:

MDSS CH GmbH  
Laurenzenvorstadt 61  
5000 Aarau  
Switzerland

This document is issued in JUNE 2022.

© 2022 – Materialise NV. All rights reserved.

Materialise, the Materialise logo and the Mimics product name are trademarks of Materialise NV.

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