# materialise

innovators you can count on

## SurgiCase Knee Planner 3.5.1

INSTRUCTIONS FOR USE - EN

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## INSTRUCTIONS FOR USE - EN

**This document contains general instructions for use for SurgiCase Knee Planner. For detailed instructions, please refer to the user manual present in the software.**

### DESCRIPTION

The SurgiCase Knee Planner is a 3D environment software which allows the surgeon to pre-operatively plan a knee arthroplasty surgery based on images of the patient.

### INDICATIONS FOR USE

#### US

The SurgiCase Knee Planner is intended to be used as a pre-surgical planner for knee orthopedic surgery. The software is used to pre-operatively plan the positioning of knee components. The SurgiCase Knee Planner allows the surgeon to visualize, measure, reconstruct, annotate and edit pre-surgical plan data. The software leads to the generation of a surgery report along with a pre-surgical plan data file which is used as input data to design the *Materialise Knee Guides and Models*.

#### EU and ROW

The SurgiCase Knee Planner software is intended to be used for simulation of surgical interventions and the planning of preoperational interventions.

The SurgiCase Knee Planner software can be used to assist in the positioning of orthopedic components.

### CONTRAINDICATIONS

None.

### WARNINGS

- The software should be operated by experienced personnel, or the output should always be reviewed and approved by a qualified surgeon.
- It is the surgeon's ultimate obligation to exercise his/her professional judgment in any decision to follow or not follow the treatment plan made using the medical device software.
- It is the surgeon's responsibility to only use the software on desktop or mobile devices satisfying the minimal requirements specified below.
- Be aware that the anatomical models shown in the software are derived from the images of the patient. If the patient's anatomy has changed significantly since the time of the images, the surgical plan should not be used.

### PRECAUTIONS

**R<sub>x</sub>only** CAUTION: Federal law (USA) restricts this device to sale by or on the order of a Physician.

### INSTRUCTIONS FOR USE

- Install  
No installation is required to use the SurgiCase Knee Planner. The SurgiCase Knee Planner is a web based application and can be accessed by browsing to the URL of the case.
- Start  
To open a case in the SurgiCase Knee Planner, browse to the appropriate case in the medical device data system and click the link in the Case Details to open the SurgiCase Knee Planner.

## MINIMAL REQUIREMENTS

### Operating Systems

Following operating systems are supported:

- Latest version of iOS
- Latest version of Mac OS
- Windows 10

### Browsers

Browser with WebGL support and WebGL enabled.

Following browsers are supported by the SurgiCase Knee Planner per operating system:

- Google Chrome on Windows, Mac OS X
- Microsoft IE on Windows OS desktop
- Microsoft Edge on Windows OS desktop
- Mozilla Firefox on Windows OS desktop
- Apple Safari on Mac OS X, iOS OSs

Cookies support

### Internet Connection

3G or higher

### Application Window Resolution

WXGA or higher

### Screen size

7 inches or more

### Memory

2 GB RAM or more

### CPU

Dual Core processor or better

### Graphic Card

Please refer to the following list describing black/white list of graphic cards:

<https://www.khronos.org/webgl/wiki/BlacklistsAndWhitelists>

## AUSTRALIAN SPONSERS

Emergo Australia Level 20 Tower II Darling Park

201 Sussex Street

Sydney NSW 2000

Australia

**And**

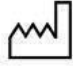
Zimmer Biomet Pty Ltd,

12 Narabang Way, BELROSE

NSW 2085

Australia

**Description of symbols which are used in the About Box of the software:**

	Legal Manufacturer
	Caution, please consult the instructions for use and the accompanying documentation
	Consult instructions for use or consult electronic instructions for use
	Date of manufacture
	Batch code
	Medical device
	Unique Device Identifier

The Instructions for Use for the SurgiCase Knee Planner is supplied in electronic form in PDF format on [www.materialise.com/electronic-instructions-for-use](http://www.materialise.com/electronic-instructions-for-use). Paper version may be requested by emailing [leaflet@materialise.be](mailto:leaflet@materialise.be) and will be provided within 7 calendar days at no additional cost.

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