Materialise OrthoView

[Instructions for Software Use - EN](#)

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Description and performance characteristics of OrthoView software

OrthoView is a leading orthopaedic digital planning solution, used in hospitals and clinics across the globe. The innovative software provides orthopaedic surgeons with access to a comprehensive library of digital templates for joint replacement, trauma procedures, limb deformity correction, spine deformity assessment and (subject to license) paediatric assessment.

Developed in conjunction with the orthopaedic community and supplied by the major PACS suppliers, OrthoView is a digital preoperative planning solution designed specifically for orthopaedic surgeons and consists of several specialist modules, offering a complete solution for orthopaedics.

The OrthoView templating process uses a technique like the techniques that many surgeons previously practiced with acetate templates over x-ray film. OrthoView can assist with planning operative procedures by scaling, planning, reduction, and templating on one or more patient images and will generate reports that can accompany patient notes.

OrthoView template wizards help you to get from an image of a patient's x-ray quickly and easily to the approximate template size needed. The visual templating features then let you quickly review multiple template sizes, against the x-ray, and obtain an accurate view of the implant size required.

Indications for use/Intended purpose

The Intended use of OrthoView is when a suitably licensed and qualified healthcare professional requires access to medical images with the intention of using such images for pre-operative planning of a potential surgical procedure.

OrthoView is indicated for use when a suitable licensed and qualified healthcare professional requires access to medical images with the intention of using such images to plan or review a surgical procedure. OrthoView provides a set of tools and templates (representing prosthetic and fixation devices) to assist the healthcare professional in planning their surgery.

Contraindications

The device is not to be used for mammography.

Warnings and recommendations

OrthoView software should only be used by trained medical professionals. To engage in the experience, we recommend that all users participate in a training offered by a Materialise expert.

As stipulated in the End User License Agreement, Materialise warrants during the Warranty Period that OrthoView shall operate substantially in accordance with the functional specifications in the Documentation. Any other warranty, whether express or implied, including but not limited to any warranty regarding fitness for purpose, safety, performance or effectiveness aspects of the medical application or device for which OrthoView is used, is disclaimed.

OrthoView displays measurements with 1 digit behind the decimal point.

For a patient or user in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices): if, during the use of this device or as a result

of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Any diagrams or pictures used in the Help and in accompanying documents associated with the product and in promotional literature are for illustrative purposes only and any readings, displays or indications may not be the same as those obtained during actual use.

Any user settable parameters in the software or on the PC must be checked and deemed acceptable by the operator before use.

Best-practice tips and precautions

Be advised that the quality and resolution of medical images can influence the result of analysis operations. For best results, we recommend that industry standards are referenced and followed.

Instructions for use

Installing OrthoView

Server Version (WK-WS)

For WK-WS installation, instructions to launch OrthoView from a server and more information about the installation process, please refer to the installation guide provided with the software.

Desktop version (WK)

You will receive a temporary OrthoView license file as an attachment to an email from OrthoView, and this file will have the extension **.OVLF** - save this file to the machine where you intend to install OrthoView. Within this email, you should also be given your order reference number (**OR number**) which is needed in the next step, as well as whenever you contact OrthoView in the future.

Register your facility for an account at www.myorthoview.com - you will only need a single account per facility, not per user. During the registration process, you will need to supply the order reference number (e.g., **OR1234-456789**). For the software version, select the latest version. Your account registration will be checked and activated by the next business day at the latest.

Once you receive an email from orthoviewsupport@materialise.co.uk informing you that your account registration has been activated, you can return to the site and login to obtain the software from the '**My Software**' section. Download this file to a PC where OrthoView is to be installed.

Decide whether you are going to use a concurrent or stand-alone license - to find out the differences between these options, check the installation guide provided with the software.

Start the installation by launching the downloaded installation file (*.exe format).

In every next dialogue box, select your preferred options and choose 'Next' to continue. Do the actual installation by choosing 'Install', a progress bar will keep you up to date about the status. If the application is installed successfully, the Finish dialogue will appear, allowing you to complete the installation (click 'Finish').

Starting OrthoView

Once the installation process has finished, run OrthoView either from the Desktop shortcut or via the Start menu, depending on the options, you chose during installation. Now install the temporary license. After that you should request a permanent license for the installation.

For alternative ways to start OrthoView please refer to the integration guide provided with the software.

Uninstall can safely be done from the Control Panel > Programs > Programs and Features menu.

Minimum system requirements

Computer	PC
Operating System Windows	Windows 7, 8, 8.1, 10. Windows 11 from OrthoView versions 7.4.7 onwards
Processor	1 GHz CPU
Memory	2GB RAM
Free Hard Disc Space	2GB free disk space
Graphics/Display	1280 x 1024 colour graphics display
(Optional) Intranet Connection to PACS	100MBit/Sec
(Optional) Internet Connection (required for case management, auto template update and anonymous stat collection features only).	2MBit/Sec

It is recommended to use our software within a hardware and/or network environment in which cyber security controls have been implemented including anti-virus and use of firewall. It is also recommended not to leave the workstation on which OrthoView is running, unattended and to ensure the workstation is password protected.

Description of Symbols used in the About Box of the software

	Legal Manufacturer
	Federal law (USA) restricts this device to sale by or on the order of trained medical professionals
	The symbol means 'Refer to Accompanying Documentation.' Wherever it appears, it is drawing your attention to the fact that you should be fully conversant with the operation and use of OrthoView and that you should be familiar with all associated documentation and that you have documented training acceptable to the authority using OrthoView
	Consult instructions for use or consult electronic instructions for use
	This symbol identifies the date of release of the software.
	This symbol identifies the Lot or Version number of the software
	Medical device
	Unique Device Identifier
	This symbol identifies the reference of part code/software release version of OrthoView
	<p>This symbol shows that OrthoView is marketed as a medical device in the European Union and that it complies with all of the applicable requirements of the Medical Device Directive (93/42 /EEC).</p> <p>The '1434' is the reference of the assessment body (Notified Body) used to substantiate the claim (in this case PCBC in Poland).</p>

Request a Printed Instructions for Use

IFU for OrthoView is supplied in electronic form in PDF format on www.materialise.com/electronic-instructions-for-use. Paper version may be requested by emailing leaflet@materialise.be and will be provided within 7 calendar days at no additional cost.

Inspection and Maintenance

OrthoView is software only - there are hence no parts requiring inspection or service.

The software does not require calibration.

Units of Measure

OrthoView displays measurements in SI units legally acceptable within the European Union. Where applicable, the user may select alternative non-standard units where medical convention, for instance, uses such units.

Explanations of functions and controls

Each control and display have associated context sensitive help. Lay the pointer over the control and an explanatory pop-up box will appear. Refer also to the system SmartHelp for an explanation of OrthoView and the sequence of operation.

Special Operating Instructions

To prevent misuse or abuse by others, protect your user access codes and do not disclose them to others. Always log off the system or otherwise lock the use of the workstation when you are away from it.

Technicians, particularly Administrators, must not attempt to alter code or settings that would be entered by Clinical Users. They must not use the software as an Operator when logged on as an Administrator in case this overwrites any Clinical User data or settings.

Operators must not attempt to provide extremes of data input for instance by pressing keys continuously or providing intentionally incorrect input, toggling IT equipment power etc.

Persons operating and installing OrthoView must not have sensory/physical impairment to a degree that the IT equipment cannot be operated effectively, and warnings and cautions cannot be discerned or acted upon.

Operators must verify that the displayed user details are those for which the plan is required. Stored plans must be controlled (reliably stored, backed up) by the operator or technician. This is particularly important where plans are saved to disk rather than direct to a PACS system.

USA – Prescription Statement (21 CFR 801.109/b/1)

Caution: USA Federal law restricts this device to sale by or on the order of a physician.

Manufacturer contact details

Approved distributors and how to obtain service and maintenance

In the first instance, please contact your Customer Support Engineer. In case of difficulty, please contact the Customer Support Department at Meridian Technique as shown below:

Manufactured by:

Meridian Technique Ltd
5 Benham Road
Southampton Science Park
Southampton
Hampshire
SO16 7QJ
Great Britain
Telephone +44 (0)2380 762 500
Email: orthoviewsupport@materialise.co.uk

<http://www.materialise.com>

Australian Sponsor:

AA-Med Pty Ltd
Level 8
1 Chandos Street
St Leonards NSW 2065
Australia
Telephone: 1300 887 807

EU Authorised Representative

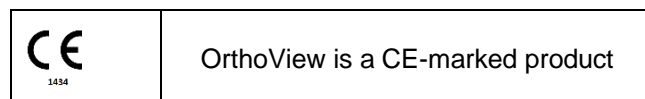
EC	REP
----	-----

Materialise NV
Technologielaan 15
Leuven
3001
Belgium

Switzerland Authorised Representative

CH	REP
----	-----

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

This document is issued in MAY 2022.

© 2022 – Materialise N.V. All rights reserved.

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