The logo for Materialise, featuring the word "materialise" in a bold, lowercase sans-serif font. Below it, the tagline "innovators you can count on" is written in a smaller, lowercase sans-serif font. The background of the page is white, with a large blue triangle on the right side and a smaller blue triangle on the left side, meeting at a diagonal line that passes through the center of the page.

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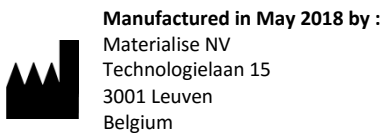
## INSTRUCTIONS FOR USE - EN

**This document contains general instructions for use for SurgiCase Knee Planner. For detailed instructions, please refer to the user manual present in the software.**

### DESCRIPTION

The SurgiCase Knee Planner is a 3D environment software which allows the surgeon to pre-operatively plan a knee arthroplasty surgery based on images of the patient.

### INDICATIONS FOR USE

The SurgiCase Knee Planner software is intended to be used for simulation of surgical interventions and the planning of preoperational interventions.

The SurgiCase Knee Planner software can be used to assist in the positioning of orthopedic components.

### CONTRAINDICATIONS

None.

### WARNINGS

- The software should be operated by experienced personnel, or the output should always be reviewed and approved by a qualified surgeon.
- It is the surgeon's ultimate obligation to exercise his/her professional judgment in any decision to follow or not follow the treatment plan made using the medical device software.
- It is the surgeon's responsibility to only use the software on desktop or mobile devices satisfying the minimal requirements specified below.
- Be aware that the anatomical models shown in the software are derived from the images of the patient. If the patient's anatomy has changed significantly since the time of the images, the surgical plan should not be used.

### INSTRUCTIONS FOR USE

- **Install**  
No installation is required to use the SurgiCase Knee Planner. The SurgiCase Knee Planner is a web based application and can be accessed by browsing to the URL of the case.
- **Start**  
To open a case in the SurgiCase Knee Planner, browse to the appropriate case in the medical device data system and click the link in the Case Details to open the SurgiCase Knee Planner.

### MINIMAL REQUIREMENTS

#### Operating Systems

Following operating systems are supported:

- Latest version of iOS
- Latest version of Mac OS
- Windows® 7 Service Pack 1 (SP1) (32-bit or 64-bit)
- Windows 10

#### Browsers

Browser with WebGL support and WebGL enabled.

Following browsers are supported by the SurgiCase Knee Planner per operating system:

- Google Chrome on Windows, Mac OS X
- Microsoft IE on Windows OS desktop
- Microsoft Edge on Windows OS desktop
- Mozilla Firefox on Windows OS desktop
- Apple Safari on Mac OS X, iOS OSs

Cookies support

#### Internet Connection

3G or higher

#### Application Window Resolution

WXGA or higher

#### Screen size

7 inches or more

#### Memory

2 GB RAM or more

#### CPU

Dual Core processor or better

#### Graphic Card

Please refer to the following list describing black/white list of graphic cards:

<https://www.khronos.org/webgl/wiki/BlacklistsAndWhitelists>

#### Australian Sponsors

Emergo Australia Level 20 Tower II Darling Park  
201 Sussex Street  
Sydney NSW 2000  
Australia

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